



## Exposure, Pre Set Times and Functions



1. Press the exposure button twice. The first time pressing, a 60 sec preparation will state and so, press immediately the second time to have the x-ray emitting.

2. At the end of the exposure, the screen will display the dose value of the exposure. At the same time the icon  flashes, which indicates that the equipment is cooling. Wait until the icon  is closed to prepare for the next exposure



**E01 - ERROR CODE**  
The exposure button was released before programmed time. Press the buttons UP/DOWN together for 3 sec. to release



**Locking the System**  
To lock the x-ray exposure, press and hold MODE button and then the TEETH button and release it. To unlock the system, do the same procedure



**mA Measure**  
To display de mA measure, press and hold TEETH button and them, press the DOWN button



**kV Measure**  
To display de kV measure, press and hold TEETH button and them, press the UP button

