

## X-RAY RAY68 AC RUNYES

### RAY68 X-RAY – ERROR CODE TABLE

Dental X-Ray Unit RAY68

70kvp - 7mA

#### 7. Error Codes

Error code				
Cod e	Error	Signal	Operation	Solution
A01	The voltage of power is lower than exposure voltage	Red alarm lamp is on/The LED display error code A01	Prohibit any operation	Waiting for automatic restoration after voltage restoration
A02	The voltage of power is higher than exposure voltage	Red alarm lamp is on/The LED display error code A02	Prohibit any operation	Waiting for automatic restoration after voltage restoration
A05	The controller and the tubehead are disconnected or the head break down	Red alarm lamp is on/The LED display error code A05	Prohibit exposure	Switch off the power then check the connection
A08	Release the key during exposure	Red alarm lamp is on/The LED display error code A08	Prohibit exposure	Press any key to recover(except SET and SP keys)

**Warning**

Normally the A1 and A2 failure means the using power voltage is not stable .We suggest the installation of a 1.5KW automatic AC Voltage Regulators.

**WARNING: THIS PROCEDURE MUST BE DONE FOR A SPECIALIZED TECHNICIAN. WE ARE NOT RESPONSIBLE FOR ANY ACIDENT OR DAMAGE DURING THE PROCEDURE**