

PORTABLE X-RAY

DIOX – ERROR CODE TABLE



SECTION 4. TROUBLESHOOTING

4.1 ERROR CODES

Error code	Error name	Cause and Handling
B.EMPTY	Low Battery error	This error code indicates the battery needs recharging. Please recharge the battery.
LOW KV/HIGH KV	kV error	This error code indicates the kV value is exceeded the tolerance range. Calibration is required.
LOW MA/HIGH MA	mA error	This error code indicates the mA value exceeded the tolerance range. Calibration is required.
BU TIMER	Back up Time error	This error code indicates the X-ray is exposed longer than 2.2 sec. Calibration is required.
FILA CUT	Filament cut error	This error code indicates the filament is broken due to the X-ray tube is being old.
HARD ERR	Hardware error	This error code indicates that there is a hardware error in the equipment.
OVER HT	H.V tank overheat error	This error code indicates temperature of H.V Tank is higher than tolerance range. Turn off the unit and cool down for 30 minutes. Refer to <i>Chapter 2.3</i> about Duty cycle information.

WARNING: THIS PROCEDURE MUST BE DONE FOR A SPECIALIZED TECHNICIAN. WE ARE NOT RESPONSIBLE FOR ANY ACIDENT OR DAMAGE DURING THE PROCEDURE