



xrddental

Procedure: How to adjust the Arm of the Z70 Intraoral Dental X-Ray

1. Locate the 4 points (A,B,C,D) at the picture 1.A. Start with the adjustment on the part A (as shows on the picture 1.A) removing the caps according with the instructions at the pictures 1.B and 1.C.

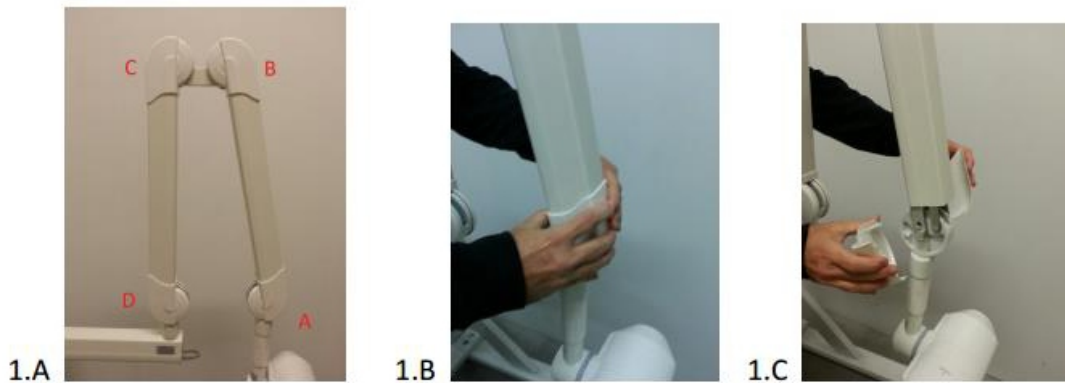

2. Apply the WD-40 as indicate on the pictures bellow (2.A, 2.B, 2.C). Clean the excess of the product and move the arm back and forward to see if the arm is smooth according your wish. In case it is put the white caps back and terminate the process, otherwise follow the next steps.

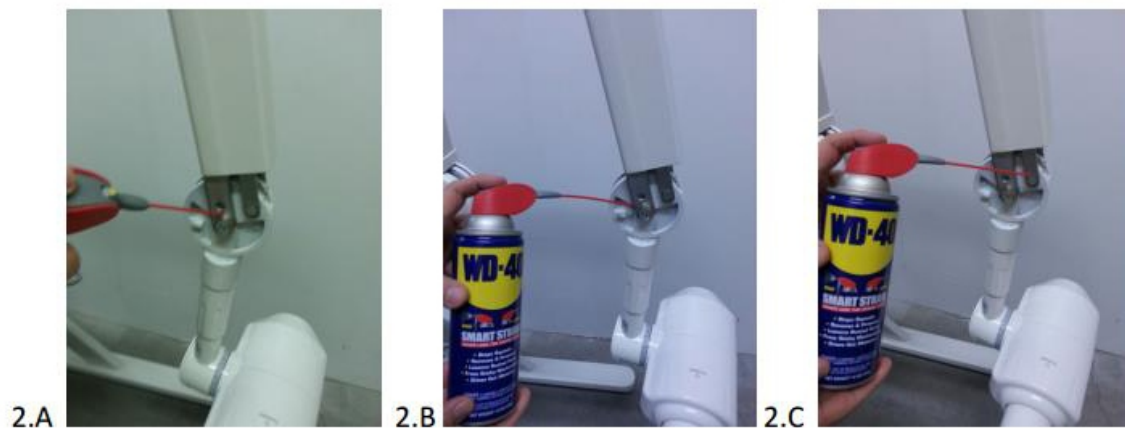

1/2


WARNING: THIS PROCEDURE MUST BE DONE FOR A SPECIALIZED TECHNICIAN. WE ARE NOT RESPONSIBLE FOR ANY ACIDENT OR DAMAGE DURING THE PROCEDURE

xrddental

3. The next step will be necessary the use of the tools (2 of each) according to the picture 3.A (sizes respectively #12, #17, #2). Following to the picture 3.B, loosen gently the threads and check if the arm is ready moving it back and forward. If it is return the caps and finish the procedure.



4. In case it is still stiff follow the steps 4.A losing both black screws, moving to the picture 4.B, loose the two threads and check if the arm is already softer. **If the arm is softer and the process is done, come back to the picture 4.A and tight the two black screws before closing it with the caps.**

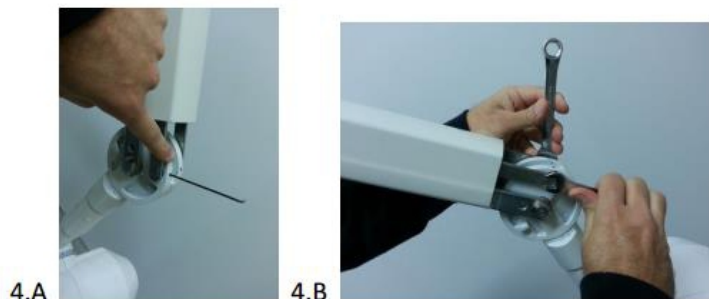

The same procedure can be done on the 4 points of the arm as showing on the picture 1.A.

Also it's important to loose it slowly and checking if it's smooth enough. We don't recommend doing it too much otherwise it will disassemble the arm, **BE CAREFUL.**

Warning: XZeal is not responsible for any accident that may occur during this process or damage on the equipment. It has to be done by a qualified technician.