

X-RAY XZEAL Z70

XZEAL Z70 X-RAY – ERROR CODE TABLE

Dental X-Ray Unit **Z70**
70kVp - 7mA

7. Error Codes

Code	Description	Signal	Operation	Solution
P1	The voltage of power is lower than exposure voltage	Red alarm lamp is on/The LED display error code P1	Prohibit any operation	Waiting for automatic restoration after voltage restoration
P2	The voltage of power is higher than exposure voltage	Red alarm lamp is on/The LED display error code P2	Prohibit any operation	Waiting for automatic restoration after voltage restoration
P3	The exposure finished and the tube is in state of cooling, pausing the functions	The LED display code P3	Prohibit exposure	Waiting for automatic restoration after pre-set time (1/60)
P4	The exposure process ended earlier than adjusted	The LED display code P4	Prohibit exposure	Waiting for automatic restoration
P5	Line released between controller and Tubehead or Tubehead broken	The LED display code P5	Prohibit exposure	Contact a specialized professional or XZeal to solutions

Warning

Normally the P1 and P2 failure means the using power voltage is not stable .We suggest the installation of a 1.5KW automatic AC Voltage Regulators.

WARNING: THIS PROCEDURE MUST BE DONE FOR A SPECIALIZED TECHNICIAN. WE ARE NOT RESPONSIBLE FOR ANY ACIDENT OR DAMAGE DURING THE PROCEDURE