

PORTABLE X-RAY

PROX – ERROR CODE TABLE



SECTION 4. TROUBLESHOOTING

4.1 ERROR CODES

| Error code | Error name | Cause and Handling |
|----------------|-------------------------|---|
| B.EMPTY | Low Battery error | This error code indicates the battery needs recharging. Please recharge the battery. |
| LOW KV/HIGH KV | kV error | This error code indicates the kV value is exceeded the tolerance range. Calibration is required. |
| LOW MA/HIGH MA | mA error | This error code indicates the mA value exceeded the tolerance range. Calibration is required. |
| BU TIMER | Back up Time error | This error code indicates the X-ray is exposed longer than 2.2 sec. Calibration is required. |
| FILA CUT | Filament cut error | This error code indicates the filament is broken due to the X-ray tube is being old. |
| HARD ERR | Hardware error | This error code indicates that there is a hardware error in the equipment. |
| OVER HT | H.V tank overheat error | This error code indicates temperature of H.V Tank is higher than tolerance range. Turn off the unit and cool down for 30 minutes. |

WARNING: THIS PROCEDURE MUST BE DONE FOR A SPECIALIZED TECHNICIAN. WE ARE NOT RESPONSIBLE FOR ANY ACIDENT OR DAMAGE DURING THE PROCEDURE